## Reusable 2-ply cotton face mask

Copyright © XD P265.89X

### ENGLISH

#### P265.89X REUSABLE 2-PLY COTTON FACE MASK

This community/barrier mask is intended for use by healthy people who do not present any clinical symptoms of viral infection and who are not in contact with people presenting such symptoms.

Wearing this mask is restricted to half a day (4 hours) and constitutes a protective barrier against possible penetration of the virus in the user's mouth and nose area or from a nearby person. It is intended to protect this area against any contact with the hands.

The face cover dimensions are complying with EU sizing and are based on certain anthropomorphic data in ISO/TS 16976-2:2015 "respiratory protective devices – Human factors – part 2: Anthropometrics". This mask is not a medical device in the sense of Regulation EU/2017/745 (surgical masks) nor is it personal protective equipment in the sense of Regulation EU/2016/425 (filtering masks type FFP2)

#### 1. CARE INSTRUCTIONS

- Machine washable on 60 degrees
- Storage condition: temperature 4°C- 40°C, humidity under 50%

#### 2. \*\*\* WARNING \*\*\*

- This product may only be used by children > 3 years due to choking hazard. Children need to be supervised if wearing this mask.
- Wearing this mask will cause a breathing resistance.
- This barrier/community mask must never be used for protection against chemicals
- This mask are not designed to protect the lungs against potential risks in the air. This mask is designed to avoid touching your mouth and nose. It is also designed to reduce the spread of saliva while talking, sneezing or coughing.
- Wearing this community/barrier mask does not exempt you from any social distance or hygiene rules.
- Do not wear this mask with heavy efforts such as working out
- Do not reuse any dirty or wet masks.
- Do not freeze this mask under any circumstances. They should be retained at a minimum of 4 degrees Celsius.

2

#### 3. USER INSTRUCTIONS

##### 3.1 How to wear the mask

##### 3.2 How to remove the mask

##### 3.3 Disposing of the mask

The mask must be disposed in a bin fitted with a plastic bag.

##### 3.4 Period of use

The mask must be washed each time when dirty, wet or poorly positioned on the face. It should not be put in a waiting position on the forehead or under the chin during and after use. During a single 4-hour period of use, the mask may be used several times subject to the fact that it is removed each time according to the instructions in 3.2. In all cases, it shall be less than 4 hours over a single day. (Equivalent to half a day).

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